Improving Healthcare with Emerging Technology

reddot award 2018
winner

“Vascular Lab in a White Box”

People with P.A.D. have greater risk

Vascular Screening Device

ARTERIOVISION™ MS-3000


CE
0120

ASI together with ABI &PWV, Completely new concept of an artery assessment device

ASI with ABI and PWV will be calculated in one measurement


The ArterioVision MS-3000 is non-invasive and rapid point care quantitative test to accurately measure 4-limb Arterial Stiffness Index (ASI), Ankle Brachial Index (ABI), and Pulse Wave Velocity (PWV) for the detection of Peripheral Artery Disease (P.A.D.).

- ASI is useful for diagnosis of arterial stiffness for each limb.
- ABI is useful for prevention and diagnosis of P.A.D.
- PWV reflects cardiovascular event risk.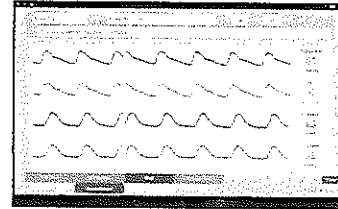



ASI is our unique & cutting edge technology.

ASI stands for Arterial Stiffness Index, which is an independent value from ABI and PWV. ASI provides a quantifier of the degree of arterial stiffness. ASI maintains correlation with age and enables quick detection of hypertension risk factors. By utilizing of ASI with ABI and PWV, diagnosis will be a groundbreaking.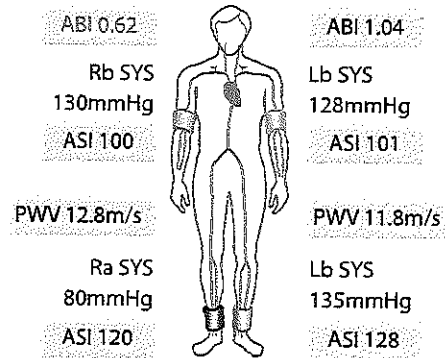

Product Specification

Classification	Class IIa
Measurement Method	Oscillometric Method
Pressure Range	0-300 mmHg
Measurement Range	40-250 mmHg
Pulse Rate Range	40-200 bpm
Power	AC100-240V 50/60Hz
Applied Part	Type BF
Accuracy	Pressure ± 3 mmHg, Pulse rate $\pm 5\%$

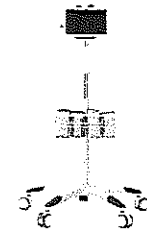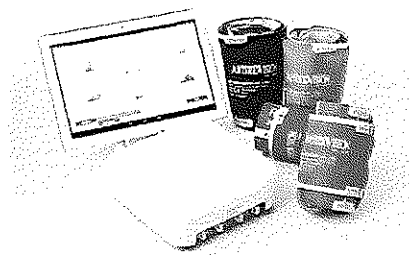Dedicated trolley (Option)

OSACHI reserves the right to change specifications without notice

E17-10arv

Contact  **OSACHI** OSACHI CO., LTD.

2-9-11, OSACHI KOHAGI, OKAYA-SHI, NAGANO 394-0085, JAPAN
 Phone. +81 266(28)0866 FAX. +81 266(28)0867
 E-Mail support@osachi.jp www.osachi.jp

Manufacturer  **SANYOSEIKO** SANYOSEIKO CO., LTD.

1435, OZAWA, SARUHASHI-MACHI, OTSUKI-SHI, YAMANASHI 409-0616, JAPAN
 Phone. +81 554(22)1036 FAX. +81 554(22)7076
 E-Mail support@sanyoseiko.co.jp www.sanyoseiko.co.jp/en

Name: thitikal

ID: 180220191

Age: 52yrs


Gender: Female

Department:

Hospital Ward:

Clinician:

Height: 152.0cm Weight: 75.0kg BMI: 32.5kg/m²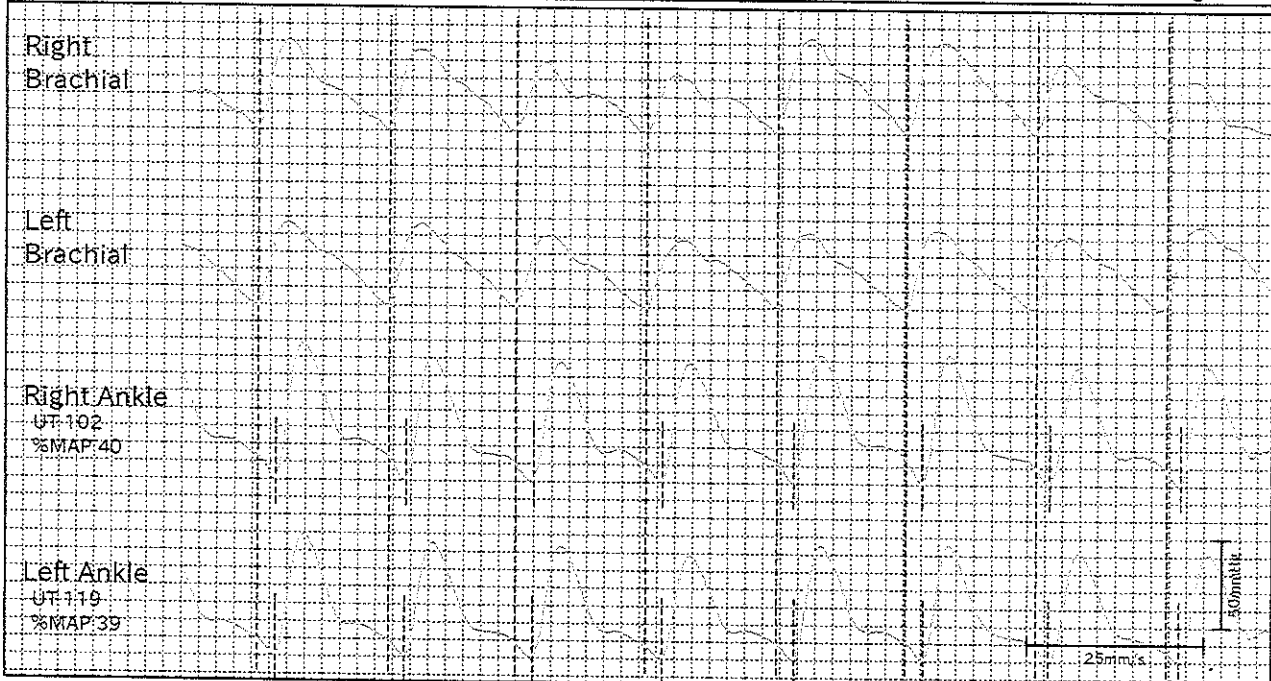

R ABI 1.12

Pulse 82bpm

L ABI 1.11

Right Brachial

BP R-L differences
7mmHg




Left Brachial

ASI 71
Systolic 125
Mean 102
Diastolic 80
Pulse Press. 45

baPWV(R)
12.5m/s

baPWV(L)
12.3m/s

ASI 45
Systolic 118
Mean 103
Diastolic 75
Pulse Press. 43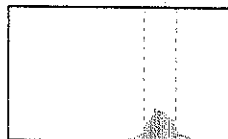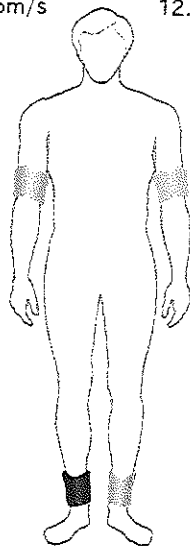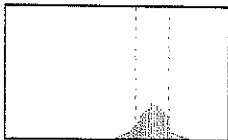


Right Ankle


ASI 104
Systolic 140
Mean 97
Diastolic 69
Pulse Press. 71

Left Ankle


ASI 113
Systolic 139
Mean 93
Diastolic 75
Pulse Press. 64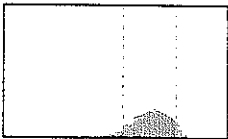


BP Unit
mmHg

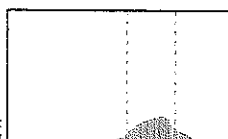ASI Graph(Female)

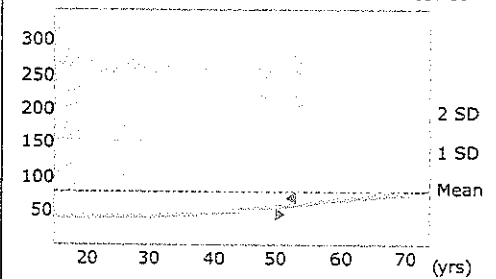ASI

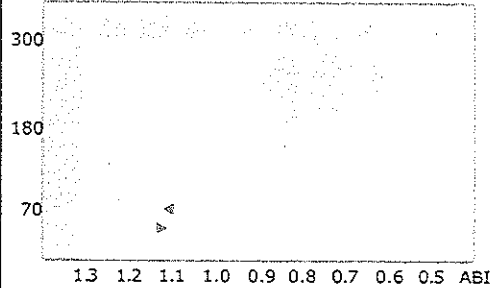Observations

[Right Brachial] Normal
[R ASI] Moderate (CAD) Risk
[R ASI] The measurement results are within the standard level of a healthy 52 and female.
[R ABI] Normal

[Left Brachial] Optimal
[L ASI] Low (CAD) Risk
[L ASI] The measurement results are within the standard level of a healthy 52 and female.
[L ABI] Normal

ARTERIOVISION

Date: 2019/02/18 16:45

thitikal

180220191

52yrs

Female

55

152.0cm

75.0kg

32.5kg/m²

R ABI 1.12

L ABI 1.11

Arterial Stiffness Test Results

Measurement Result

Right Brachial

ASI 71

SYS 125

DIA 80


Left Brachial

ASI 45

SYS 118

DIA 75

Image of ASI and a blood vessel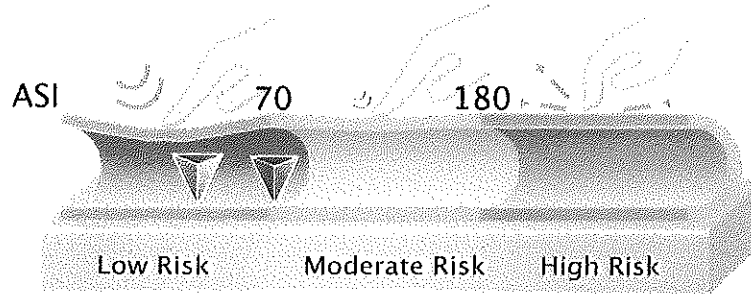

This is just a graphical illustration of an ASI.

[Right Brachial] Normal
 [Left Brachial] Optimal
 [R ASI] Moderate (CAD) Risk
 [R ASI] The measurement results are within the standard level of a healthy 52 and female.
 [L ASI] Low (CAD) Risk
 [L ASI] The measurement results are within the standard level of a healthy 52 and female.
 [R ABI] Normal
 [L ABI] Normal

Right Ankle

ASI 104

SYS 140

DIA 69

Left Ankle

ASI 113


SYS 139

DIA 75

Arterial Blood Flow Test Results

Comment

Image of ABI and a blood vessel

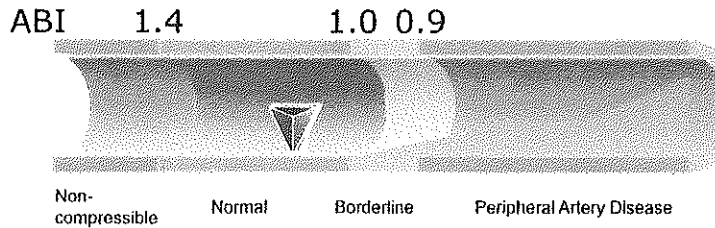This is just a graphical illustration of an AB

Blood pressure is normal but slight arterial stiffness. Regular exercise is recommended such as walking, running 10-30min, if cycling 40 min should be enough. On a bicycle machine ergometer should be set between 40-60W, rotation between 60-80 and duration of 10-30 min, Stretching and exercise is good but never over do it and proceed at a pace comfortable to you.

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MEDICIA INSTRUMENT SERVICE CO., LTD.